

Repel[®] Myco|Clean

NAIL PATCH

Hydrogel patch for nails with fungal infection

- Prevents the growth of fungal infection by changing the nail's microenvironment.

INDICATIONS

The hydrogel nail patch **REPEL MYCOCLEAN NAIL PATCH** provides continuous moisturizing effect, promotes improved appearance of discolored and deformed nails, gently smooths the outer layers of the nail plate, and prevents the growth of fungal infection by changing the nail's microenvironment.

DOSAGE

Apply for 6 to 8 hours, usually during sleep, for a period of 3 to 6 months.

INSTRUCTIONS FOR USE AND APPLICATION

INSTRUCTIONS FOR USE

1. Remove any nail polish before use. Clean and dry the area of application of patch.
2. Cut to open the pouch and take out the nail patch.
3. Wash hands and dry thoroughly as skin oils and moisture may reduce adhesion.
4. Follow the application instructions shown below.
5. Clean the nails after removing the patch.

APPLICATION INSTRUCTIONS

1. Peel off the protective films ① and ② in numerical order.
2. Apply the hydrogel patch on the nail and remove the release paper ③
3. Cover the sides of the finger firmly.
4. Fold the remaining adhesive part in order to cover the fingertip.



CAUTION

- For external use only.
- For Adult Use Only.
- Avoid contact with eyes, mouth, mucous membrane, wounds or inflamed skin. Do not use if you experienced any adverse reaction to any of the patches before.
- Do not use if you are allergic to any of its ingredients.
- Stop using if your skin turns reddish, swollen or itchy while using or after use.
- If any of the side effects become serious, or if you notice any side effects not listed in this leaflet, please consult a doctor or pharmacist.

HOW TO STORE

- Store in a cool, dry place and air-tight container at room temperature.
- Keep out of reach of children or toddlers.
- Reseal the zip-lock well to prevent the remaining patch in the foil pouch from drying.

INGREDIENTS: UREA, FUMARIC ACID, 1,3-BUTYLENE GLYCOL.

VERSION: 00

DATE: 18.12.2019



UNI-PHARMA KLEON TSETIS Pharmaceutical Laboratories S.A.

14th km National Road 1, GR-145 64 Kifissia, Athens - Greece, Tel.: +30-210-8072512, www.uni-pharma.gr

Manufacturer: Wooshin Labottach Co., Ltd. (Songdo BRC SMART Valley B-109, 914, 915, 1013, 1014, 1015, 1018, 1019) 30, Songdomirae-ro, Yeosu-gu, Incheon City, Republic of Korea.

EC REP Authorised Representative: Wooshin Lapache d.o.o Il Industrijska cesta 1J, 1290 Grosuplje, Slovenia